


#### 4 Locating the Stimulator

The Charger will beep continually as it searches for the Stimulator.

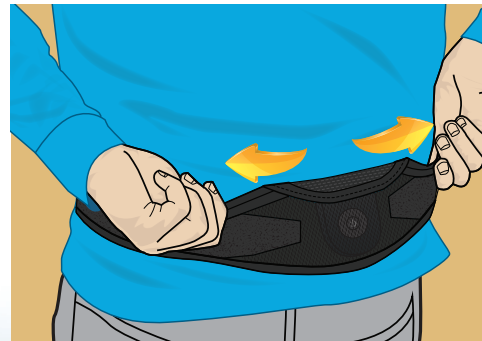


To locate the Stimulator, place the Charger over the implant area and move it around until the beeping stops.

#### 5 Charging Tips

Occasionally check the Charger to ensure that it is centered over the Stimulator. Keeping the Charger centered over the Stimulator will ensure the shortest charging time.

The Indicator Light may turn amber. This is NORMAL and you may continue charging.

If the Charger requires re-charging, it will beep slowly regardless of Stimulator alignment.

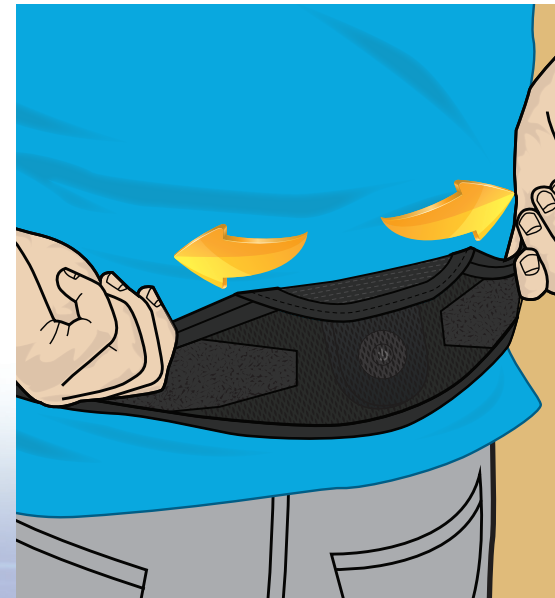

Keep charging until you hear the **DOUBLE BEEP**, indicating that the Stimulator is fully charged.


You can also check the stimulator battery indicator on the Remote Control to ensure that you have completely recharged your Stimulator's battery.

**WARNING:** While charging your IPG, the Charger will become warm. This is considered normal. Handle the Charger with care. **DO NOT USE THE CHARGER WHEN SLEEPING.** This may result in a burn. Failure to use the Charging Belt may also result in a burn. If you experience discomfort or pain, cease charging and contact Boston Scientific Neuromodulation.

#### 6 Charging Complete

Remove the Charging Belt.

Return your Charger to the Base Station.

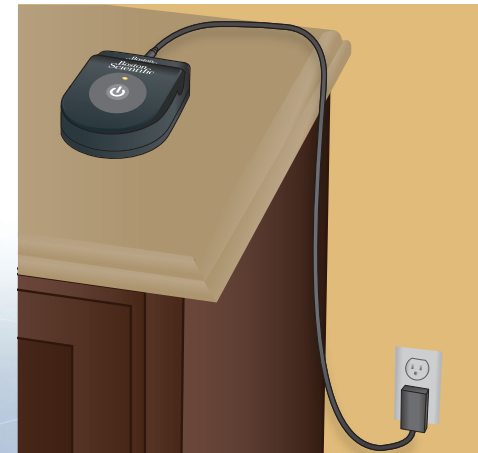

Not intended as a substitute for the provided *Charging System Handbook*. The contents of the handbook must be fully understood before using your charging system.

All cited trademarks are the property of their respective owners. **CAUTION:** The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Precision is a registered trademark of Boston Scientific Corporation or its affiliates.

**Boston Scientific**  
Advancing science for life™

© 2018 Boston Scientific Corporation or its affiliates. All rights reserved.

9055582-003  
Content: 90914728 REV AD  
EN

**CE** 0123 Authorized to affix the CE mark in 2015

**Boston Scientific Neuromodulation Corporation**  
25155 Rye Canyon Loop  
Valencia, CA 91355

866.360.4747 Toll-free  
877.464.2940 Fax  
ControlYourPain.com

**PRECISION™**  
Spinal Cord Stimulator Systems

**Boston Scientific**  
Advancing science for life™


Charging System  
Quick Start Guide



SC-5312

## 1 Power Supply

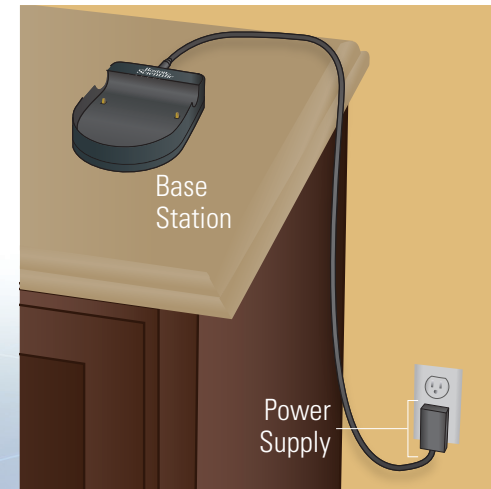

Plug the Power Supply into an outlet and connect to the Base Station.



## 2 Charging the Charger

Place the Charger in the Base Station. When your Charger is fully charged, the light will turn green.



-  Green light = a full charge
-  Amber light = a partial charge

*No light indicates that a connection is loose.*



## 3 Using the Belt

Place the Charger in the pouch of the Charger Belt and push the Power Button. Close flap.




*Be sure that the Charger is facing outward.*

To open, undo Velcro® strap and pull through belt loop.

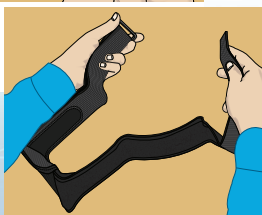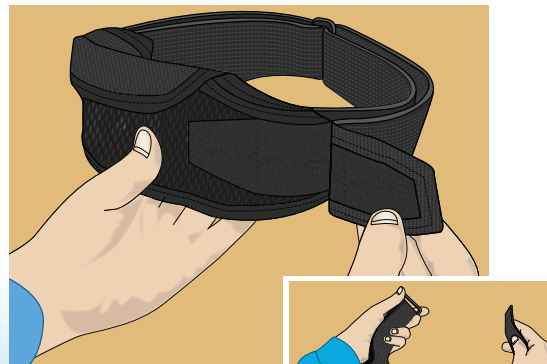

Wrap the Charger Belt around your waist and insert the strap back through the loop.

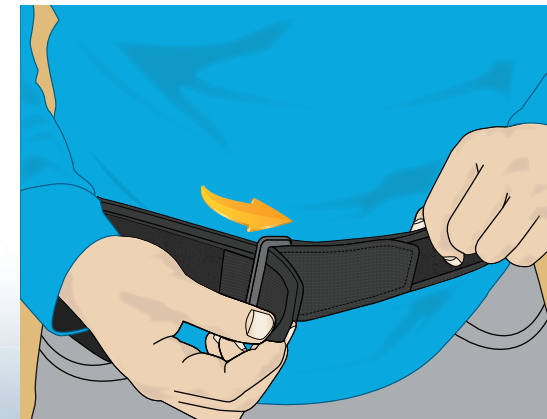

*Be sure that the Charger is directly over the Stimulator.*

Pull the strap in the opposite direction and secure the Velcro.



If necessary, you can adjust the other side.

