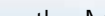
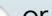

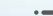
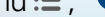







Notes

5 Changing Schedules



- From the Main Menu , press  or  to select Schedules.
- Press  to enter the Schedules menu .
- Press  again to enter the Change Schedule menu .
- Press  or  to select the desired Schedule.
- Press  to activate the selected Schedule.

6 Recording Therapy Ratings



- From the Main Menu , Press  or  to select Therapy Rating .
- Press  to enter the Therapy Rating Screen.
- Press  or  to select the desired Rating.
- Press  to record the desired Rating.

7 Recharging the Remote Control



Your Remote will display a message reminding you to recharge whenever the battery power becomes low.

- Connect the RC to the USB Power Supply provided with the RC.



- Connect the power plug end of the USB Power Supply to a power outlet.

Not intended as a substitute for the provided handbooks (Remote Control, Information for Patients, or Patient Trial). The contents of the handbook must be fully understood before using your Boston Scientific SCS System.

All trademarks are the property of their respective holders

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Freelink is a registered trademark of Boston Scientific Corporation or its affiliates.

Boston Scientific
Advancing science for life™

Legal Manufacturer

Boston Scientific Neuromodulation Corporation
25155 Rye Canyon Loop
Valencia, CA 91355 USA
(866) 789-5899 in US and Canada
(661) 949-4000, (661) 949-4022 Fax
(866) 789-6364 TTY
www.bostonscientific.com
Email: neuro.info@bsci.com

AUS Australian Sponsor Address

Boston Scientific (Australia) Pty Ltd
PO Box 332
BOTANY
NSW 1455
Australia
Free Phone 1800 676 133
Free Fax 1800 836 666

EC REP EU Authorized Representative

Boston Scientific Limited
Ballybrit Business Park
Galway, Ireland
T: + 33(0)1 39 30 97 00
F: + 33(0)1 39 30 97 99

© 2018 Boston Scientific Corporation or its affiliates. All rights reserved.

92253942-01
Content: 92253943 REV A
3-300645_NOV2018

FREELINK™
TECHNOLOGY

Remote Control
Quick Start Guide



Model SC-5261
Model SC-5260

Rx ONLY

CAUTION: Federal law restricts this device to sale, distribution and use by or on the order of a physician.

92253942-01
Content: 92253943 REV A

Boston Scientific
Advancing science for life™

1 Remote Control Buttons



Therapy refers to patterns of stimulation used to reduce the awareness of pain.

Home Screen Display



Note: You will see the icon for the Stimulator battery only if you have a rechargeable Stimulator.

Note: The Home Screen displayed is dependent on the therapy prescribed by your healthcare provider.

2 Getting Started



- 1** Unlock/ Lock
 - 2** Start/Stop Therapy
 - 3** Decrease Increase Stimulation Strength
 - 4** Main menu
- The settings displayed are dependent on your Stimulator and the therapy configured by your healthcare provider.

3 Accessing Programs

If your healthcare provider has configured Programs that can be changed, there are two ways to view the Programs menu:



- Press the Programs button on the left side of the Remote.
- From the Main Menu , select the Programs menu and press .

4 Changing Programs



- 1** From the Programs Menu press to enter the Change Program menu .
- 2** Press to select the desired Program.
- 3** Press to activate the selected Program.

My Healthcare Provider

Name

Phone

Hospital