



“I got to the point where I didn’t even want to golf anymore. I’d go golf with my buddies and I was afraid if I’d swing that club, I’d leak a little bit. I got to the point where I didn’t do any activity that was strenuous.”

— Gary

1. Chapple C, Milsom I. Urinary incontinence and pelvic prolapse epidemiology and pathophysiology. In: Wein AJ, Kavoussi LR, Novick AC, et al. (eds). *Campbell-Walsh Urology*. 10th ed. Philadelphia, PA: WB Saunders Elsevier; 2012:1871-95.
2. Markland AD, Goode PS, Redden DT, et al. Prevalence of urinary incontinence in men: results from the National Health and Nutrition Examination Survey. *J Urol*. 2010 Sep;184(3):1022-7
3. Data on file with Boston Scientific and based on market research by Dymedex.
4. Van der Aa F, Drake MJ, Kasyan GR, et al. The artificial urinary sphincter after a quarter of a century: a critical, systematic review of its use in male non-neurogenic incontinence. *Eur Urol*. 2013 Apr;63(4):681-9.
5. Sandhu, J. Treatment options for male stress urinary incontinence. *Nat Rev Urol*. 2010 Apr;7(4):222-8.
6. Montague DK. Artificial urinary sphincter: long-term results and patient satisfaction. *Adv Urol*. 2012;2012:835290.
7. Litwiller SE, Kim KB, Fone PD, et al. Post-prostatectomy incontinence and the artificial urinary sphincter: a long-term study of patient satisfaction and criteria for success. *J Urol*. 1996 Dec;156(6):1975-80.
8. AMS 800™ Urinary Control System Operating Room Manual. American Medical Systems, Inc. 2017.
9. AMS 800™ Urinary Control System Instructions for Use. American Medical Systems, Inc. 2017.
10. Bauer RM, Grabbert MT, Klehr B, et al. 36-month data for the AdVance XP™ male sling: results of a prospective multicenter study. *BJU Int*. 2017 Apr;119(4):626-30.
11. Welk B, Herschorn, S. The male sling for post-prostatectomy urinary incontinence: a review of contemporary sling designs and outcomes. *BJU Int*. 2012 Feb;109(3):328-44.
12. AdVance™ XP Male Sling System Instructions for Use. Boston Scientific Corporation. 2018.
13. Sturm RM, Guralnick ML, Stone AR, et al. Comparison of clinical outcomes between “ideal” and “nonideal” transobturator male sling patients for treatment of postprostatectomy incontinence. *Urology*. 2014 May;83(5):1186-8.
14. Suskind AM, Bernstein B, Murphy-Setzko M. Patient-perceived outcomes of the AdVance sling up to 40 months post procedures. *Neurourol Urodyn*. 2011 Sep;30(7):1267-70.

AMS 800™ Urinary Control System

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.
 Your doctor is your best source for information on the risks and benefits of the AMS 800 Urinary Control System. Talk to your doctor for a complete listing of risks, warnings and important safety information.

The AMS 800 Urinary Control System is intended for use in the treatment of male stress urinary incontinence (intrinsic sphincter deficiency) following prostate surgery.

Men with diabetes, spinal cord injuries or skin infections may have an increased risk of infection. Some AMS 800 devices contain an antibiotic (InhibiZone™ Antibiotic Surface Treatment). The device may not be suited for patients who are allergic to the antibiotics contained within the device (rifampin, minocycline or other tetracyclines) or have systemic lupus.

Potential risks may include: device malfunction/failure leading to additional surgery, wearing away/loss of tissue (device/tissue erosion), inability to urinate (urinary retention), infection, and pain/soreness. MH-545611-AA

AdVance™ XP Male Sling System

Your doctor is your best source for information on the risks and benefits of the AdVance™ XP Male Sling System. Talk to your doctor for a complete listing of risks, warnings and important safety information.

The AdVance™ XP Male Sling System is intended for the treatment of male stress urinary incontinence (SUI). Potential risks may include inability to urinate (urinary retention), return to incontinence, infection, erosion, and pain. MH-557011-AA

All images are the property of Boston Scientific. All trademarks are the property of their respective owners.

Boston Scientific Corporation
 300 Boston Scientific Way
 Marlborough, MA 01752
 www.bostonscientific.com

©2018 Boston Scientific Corporation
 or its affiliates. All rights reserved.
 MH-541205-AB DEC 2018



CONTENT IS PROVIDED BY BOSTON SCIENTIFIC. BOSTON SCIENTIFIC IS DEDICATED TO TRANSFORMING LIVES THROUGH INNOVATIVE MEDICAL SOLUTIONS THAT IMPROVE THE HEALTH OF PATIENTS AROUND THE WORLD.



Fix your incontinence




**COUGH.
 SNEEZE.
 LAUGH.
 LEAK.**

It can be humiliating and uncomfortable when you leak urine. The good news is, if your leakage is frequent or becomes overwhelming, you have options.

What is stress urinary incontinence?

Incontinence is defined as any involuntary leakage of urine.¹ Male stress urinary incontinence (SUI) is usually caused by a damaged or weakened sphincter. The sphincter is the circular muscle that controls urine flow out of the bladder. When damaged, this muscle cannot squeeze and close off the urethra; the result is urine leakage.

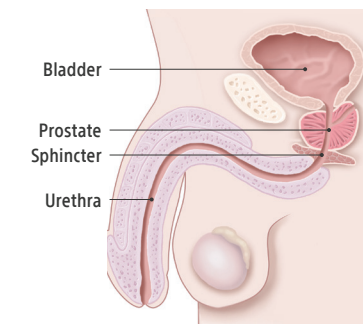

**Nearly
 1 in 10**
 American men have urinary incontinence with rates rising in men over 60 years old.²

What causes male SUI?

SUI is a common side effect of prostate cancer treatment, such as surgery (radical prostatectomy) or radiation. It can also be a symptom from enlarged prostate (BPH) surgery, pelvic trauma or a neurological condition such as spina bifida.³

**AMS 800™ Urinary Control System and
 AdVance™ XP Male Sling System**

WHAT ARE MY TREATMENT OPTIONS?

There is good news – most cases of stress urinary incontinence can be cured or improved.⁴ Your doctor may recommend changes to your daily routine, such as avoiding caffeine and alcohol, limiting fluid intake and following a voiding schedule.⁵

Some men cope by using absorbent pads and protective undergarments. Others use special devices to try to prevent urine leakage such as penile clamps, and internal and external penile catheters.

Talk to a urologist specializing in male incontinence about long-term treatment options including the AMS 800™ Urinary Control System and the AdVance™ XP Male Sling System.

AMS 800™ Urinary Control System


Considered the gold standard treatment for male SUI, the AMS 800 Urinary Control System provides proven, discreet bladder control.⁶ 90% of patients reported satisfaction with the AMS 800.⁷

The AMS 800 System is a three-part urinary control system contained completely inside the body.⁸

1. Control pump is implanted in the scrotum
2. Inflatable cuff is placed around the urethra
3. Saline-filled balloon (PRB) is usually implanted in a natural open space next to the bladder

How it works⁸

The AMS 800 System is filled with saline and uses the fluid to open and close the cuff surrounding the urethra. When you need to urinate, you squeeze and release the pump in the scrotum several times to remove fluid from the cuff. When the cuff is empty, urine can flow out of the bladder. The cuff automatically refills in a few minutes squeezing the urethra closed to restore bladder control.

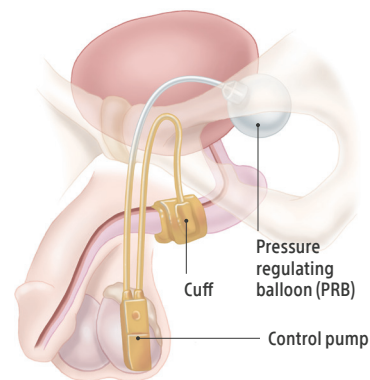


AdVance™ XP Male Sling System

The AdVance XP Male Sling is placed in the body and acts as a hammock, repositioning and supporting the urethra to help restore bladder control. Most patients (up to 89.4%) have long-term success with the AdVance XP Male Sling and are classified as cured or improved.¹⁰

How it works

The AdVance XP Sling is designed to stop leakage by supporting the sphincter muscle. The sling is a strip of soft mesh placed inside the body and is designed to move the sphincter back to its natural position to help restore bladder control. The AdVance XP Sling works on its own to restore continence, requiring no action by the patient to function.

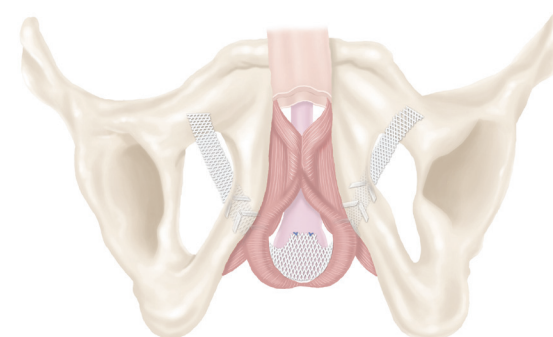

Could an AMS 800 System be right for you?

- Designed to treat male SUI following prostate surgery⁹
- Mimics a healthy sphincter, allowing you to urinate when desired⁹
- Offers most men with a weakened sphincter muscle the ability to achieve continence⁴
- Placed entirely inside the body, it is undetectable to others
- Requires good cognitive ability and manual dexterity⁹

Possible side effects include, but are not limited to: device malfunction or failure, which may require revision surgery, erosion of the urethra in the cuff area, urinary retention and/or post-operative pain.⁹ Talk to your doctor for a complete list of risks, warnings and safety information.

Recommended by patients⁷

92%

or more would have the device placed again

96%

or more would recommend the device to a friend

Could an AdVance XP Male Sling be right for you?

- Minimally invasive solution for male SUI
- Placed entirely inside the body, it is undetectable to others
- High success achieved in patients with mild to moderate SUI¹⁰
- Most patients are continent immediately following the procedure¹¹
- It operates on its own to restore continence¹¹

Possible side effects include, but are not limited to: device failure, urinary retention, post-operative pain, and/or urge incontinence.¹² Talk to your doctor for a complete list of risks, warnings and safety information.

Recommended by patients*

92%

of patients would undergo the AdVance Sling procedure again¹³

94%

would recommend the AdVance Sling procedure to a friend¹⁴

*Patient satisfaction rates reflect research results with use of the AdVance Male Sling

Take the next steps

Do not let your SUI stop you from doing the things you love in life. Find your happiness. Restore your normalcy. Renew your confidence.

Visit FixIncontinence.com to:

- Assess your bladder leakage and get your score
- Find a specialist near you
- Hear real stories from people just like you

Email us at MHPatientEducation@bsci.com or call 1-844-433-2873. We'll answer your questions and connect you with resources in your area.

